


FJORD - SALIVA COLLECTION TEST KIT FOR COVID-19


THE ONLY SALIVA COLLECTION SYSTEM THAT OFFERS **VIRAL LOAD PRE-CONCENTRATION** OF SALIVA SAMPLES TO ENHANCE VIRUS DETECTION ACCURACY

- Self-Administered
- Mitigate Infection Risk
- Storage & Transport At Ambient Temperature with Inactivating VTM

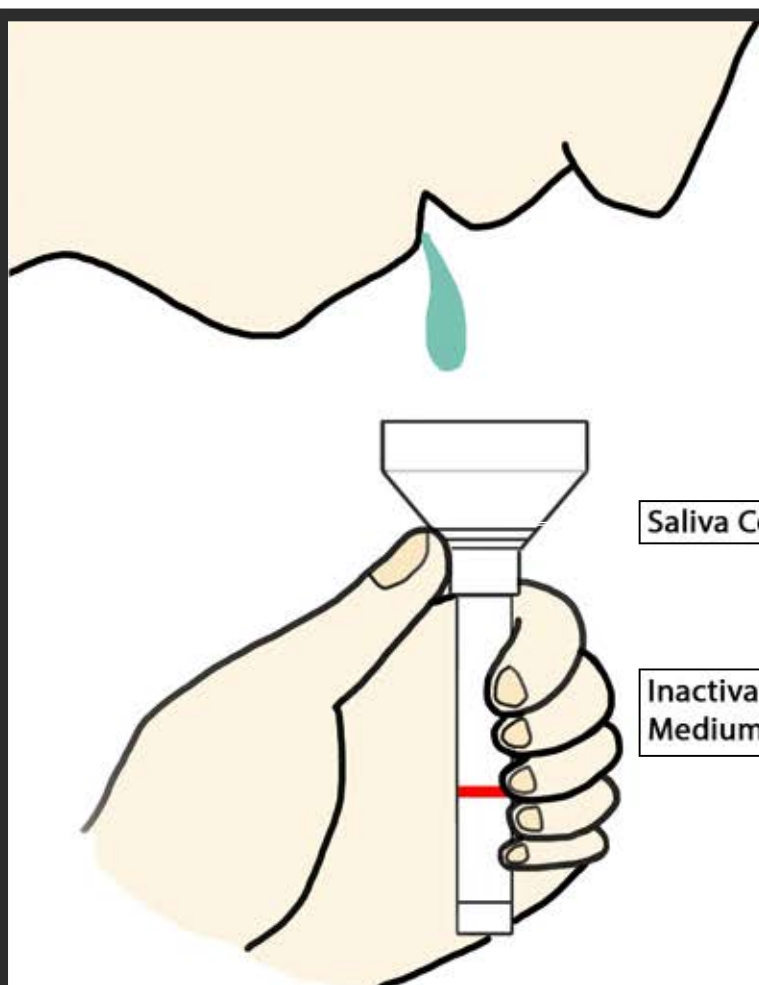

Saliva Collector Funnel

Inactivating Transport Medium

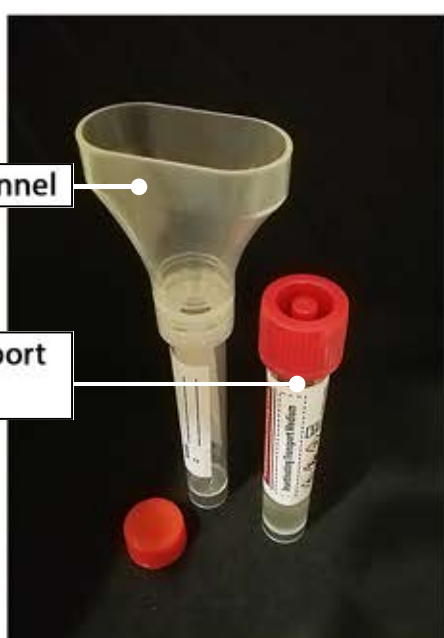

A pain-free and non-invasive method for the collection of RNA samples for Covid-19 testing

Ordering Information:

- ☐ FJORD-Saliva Collection Test Kit (Code: TM03-0300/SC)
- ☐ Product : SEDIPREP® PS 5.0 (Code: PS50-0300)







www.fjorddiagnostics.com



MDSS GmbH
Schiffgraben 41
30173 Hannover, Germany



HOW TO COLLECT A SALIVA (SPIT) SAMPLE WITH THE FJORD-SALIVA COLLECTION TEST KIT

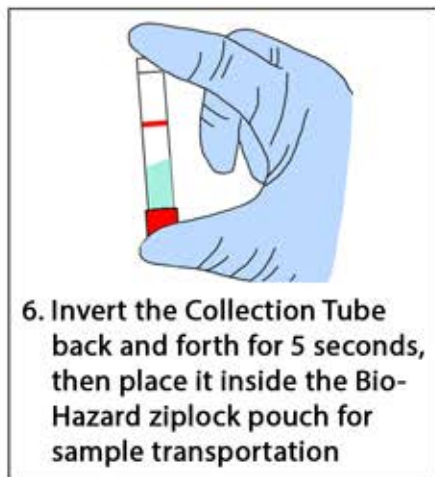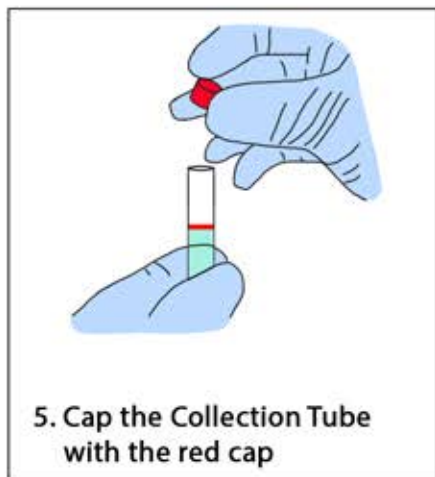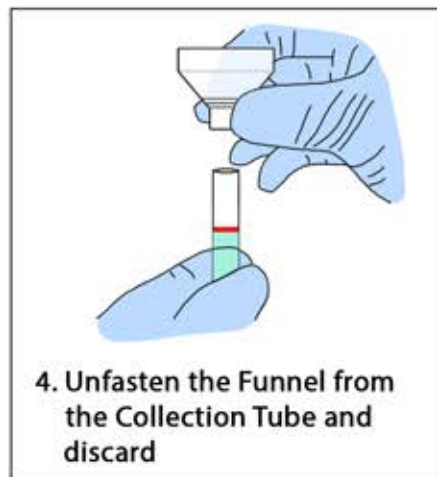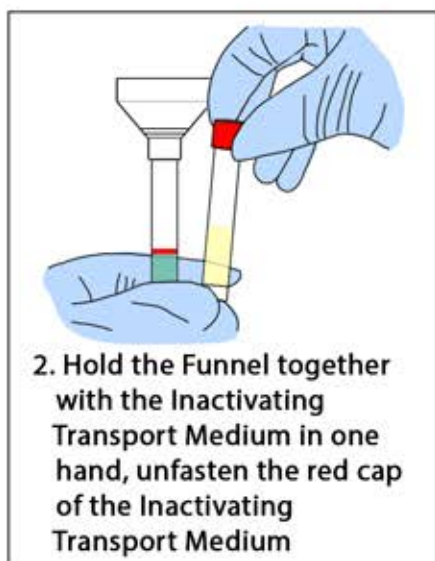


Viral Load Pre-Concentration of saliva mixture with SEDIPREP® device prior to RTK-Ag/RT-PCR analysis, for increased analytical sensitivity of virus detection

Step (i)




(i) Pour entire Saliva mixture sample into the SEDIPREP® Container Unit

Step (ii)

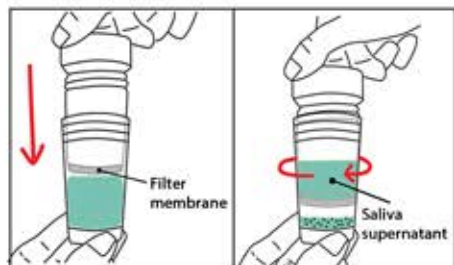

(ii) Slid the SEDIPREP® Filter Unit axially into the SEDIPREP® Container Unit and fasten clock wise direction to remove unwanted Saliva supernatant that has entered into the SEDIPREP® Filter Unit chamber

Step (iii)

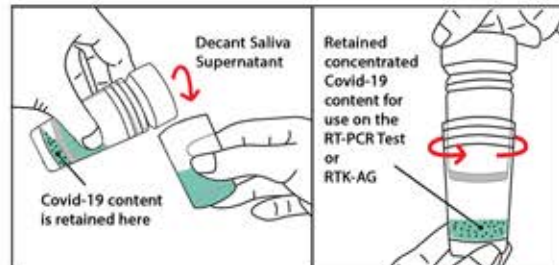

(iii) Decant the Saliva supernatant and unfasten the SEDIPREP® Filter Unit from the SEDIPREP® Container Unit. The retained concentrated Covid-19 content at the bottom of the SEDIPREP® Container Unit for use on the RTK-Ag or RT-PCR Test