

CERVICAL CANCER SCREENING WITH SEDIPREP® URINE HPV

Improved detection sensitivity with SEDIPREP® Urine HPV viral load pre-concentration

URINE HPV

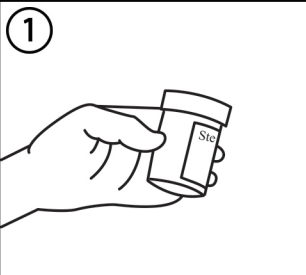
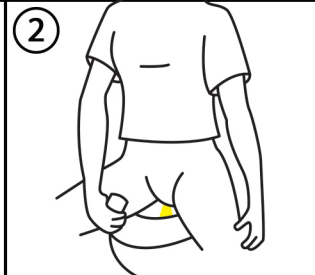
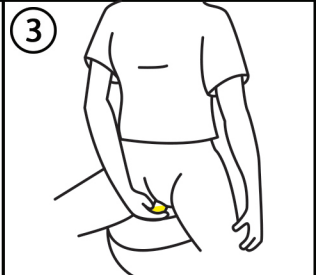


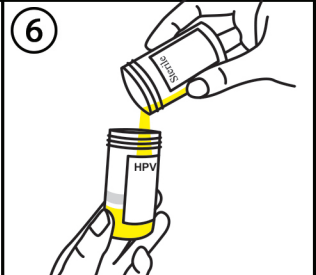
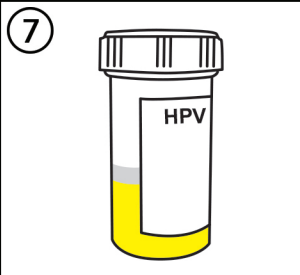
Women's Preferred Type of Sample Collection



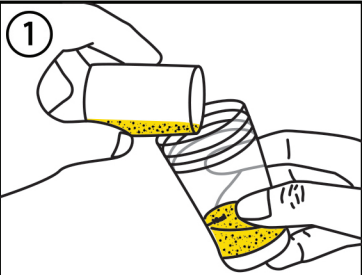
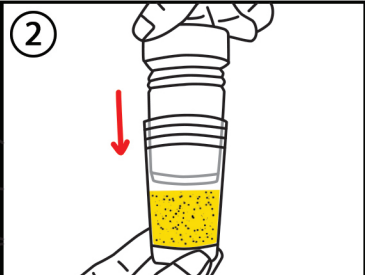
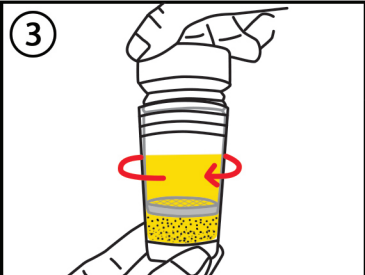
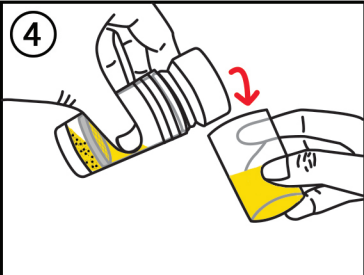
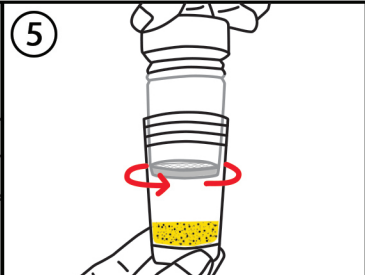

- Ease of urine sample collection
- Appeal to women participation in cervical cancer screening program
- Self-administered, no embarrassment, no pelvic examination
- No discomfort, non-invasive and pain free

Storage & Transport of Urine Sample with FJORD HPV-Sampler™

Women's preferred sample type of collection

| | | | | | | |
|--|---|---|--|---|---|---|
|  |  |  |  |  |  |  |
| <p>Get a sample container.</p> | <p>Pass the initial urine out.</p> | <p>Collect mid stream into the container.</p> | <p>Clean the soiled area.</p> | <p>Hand over the sample to the incharge.</p> | <p>Transfer urine sample into the Urine HPV Sampler™ vial up to the marking line. Urine sample mixture can be stored at room temperature for 3 weeks.</p> | <p>Transport to laboratory for SEDIPREP® viral load pre-concentration for HPV molecular analysis.</p> |

SEDIPREP® HPV Viral Load Pre-Concentration

| | | | | | |
|--|--|---|---|---|---|
|  |  |  |  |  |  |
| <p>Pour urine sample into the SEDIPREP® Container Unit, up to the calibrated line marking.</p> | <p>Slid the SEDIPREP® Filter Unit axially into the Container Unit.</p> | <p>Fasten the SEDIPREP® Filter Unit into the Container Unit, in a clockwise screw capping action up to maximum.</p> | <p>Decant the filtrate / supernatant that has entered into the SEDIPREP® Filter Unit during the clockwise fastening step.</p> | <p>Unfasten the SEDIPREP® Filter Unit from the Container Unit in the counter-clockwise uncapping action, and discard.</p> | <p>The balance concentrated urine sediment retained in SEDIPREP® Container Unit can now be used for HPV molecular analysis.</p> |