

Tissue adhesion strength ?

Not all immunohistochemistry
microscope slides are made equally.....



The **IHC-Link™ immunohistochemistry microscope slides** from FJORD DIAGNOSTICS are produced under stringent conditions to ensure every slide made, is consistently and uniformly coated across the full surface area of the slide.

IHC-Link™ Microscope Slides are intended for mounting formalin-fixed, paraffin-embedded tissue sections used in immunohistochemistry

Instructions for use :

Tissue sections of approximately 4 µm should be floated on distilled water in a water bath preheated to a maximum temperature of 45 °C.

Mount the tissue section on the coated glass area with the white frosted end. The tissue sections should be mounted on the slides as flat and wrinkle-free as possible. Following the tissue mounting, dry the slides completely at room temperature by draining them vertically before heating them in an oven for 20-60 min at 56-60 °C.



Ordering information :

Product : IHC-Link™ Microscope Slides

Art No. : 31SA-0050

Pack Size : Box of 50 slides

